


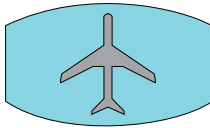

**POWER CARTRIDGE/  
AIRLINE TRAVEL TABLE**



Continuous Flow  
Pulse Flow



**CONTINUOUS FLOW**

		FLIGHT LENGTH (in Hrs)*									
		1.0	2.0	3.0	4.0	5.0	6.0	7.0	8.0	9.0	10.0
SETTING	0.5	1	1	1	1	2	2	2	2	3	3
	1.0	1	1	1	2	2	2	2	3	3	3
	1.5	1	1	1	2	2	2	3	3	3	4
	2.0	1	1	2	2	3	3	3	4	4	5
	2.5	1	2	2	3	3	4	4	5	5	6
	3.0	1	2	3	4	4	5	6	7	7	8

**ESTIMATED # OF POWER CARTRIDGES NEEDED**

\*Does not include ground time and layovers.

**PULSE FLOW**

		FLIGHT LENGTH (in Hrs)*									
		1.0	2.0	3.0	4.0	5.0	6.0	7.0	8.0	9.0	10.0
SETTING	1.0	1	1	1	1	2	2	2	2	3	3
	1.5	1	1	1	1	2	2	2	2	3	3
	2.0	1	1	1	2	2	2	2	3	3	3
	2.5	1	1	1	2	2	2	3	3	3	4
	3.0	1	1	1	2	2	2	3	3	3	4
	3.5	1	1	2	2	2	3	3	3	4	4
	4.0	1	1	2	2	2	3	3	4	4	4
	4.5	1	1	2	2	2	3	3	4	4	4
	5.0	1	1	2	2	3	3	4	4	4	5
	5.5	1	1	2	2	3	3	4	4	5	5
	6.0	1	1	2	2	3	3	4	4	5	5

**ESTIMATED # OF POWER CARTRIDGES NEEDED**

\*does not include ground time and layovers

For more information regarding traveling with your Eclipse®, or for longer flight times, please visit the SeQual® website.

[www.sequal.com](http://www.sequal.com)

**For Reorder:  
Eclipse® Power Cartridge PN: 7082**

Each airline has their own requirements for traveling with oxygen.

\*Contact your airline at least 48 hours prior to departure to inform them you will be traveling with the Eclipse.



**The Smart Choice.**

[www.sequal.com](http://www.sequal.com)

800.826.4610

PN: 5121  
Rev. A